

oncoFISH® cervical – Code 1003

Performing Lab: Ikonisys, Inc.
5 Science Park, Suite 1NW
New Haven, CT 06511

Collection: Liquid-based Pap specimen – ThinPrep®

- Submit minimum of 5mL of liquid-based Pap specimen collected in Preservcyt®, an alcohol-based solution that serves as a preservative, transport and antibacterial medium for gynecologic samples. Label the Preservcyt vial with the patient's name, unique identifier, date and source.
- Ship at room temperature. Do not freeze the specimen. Exposure of the specimen to acids, strong bases, or extreme heat should be avoided.
- Specimen must be accompanied by a laboratory requisition giving the patient's name, age, unique identifier, clinician, clinical history, procedure location, specimen type submitted, time and date.
- The reason for referral is required with specimen submission.

Collection: Liquid-based Pap specimen – SurePath™

- Submit total remnant in tube and vial. Label both the tube and vial with the patient's name, unique identifier, date and source.
- Ship at room temperature. Do not freeze the specimen. Exposure of the specimen to acids, strong bases, or extreme heat should be avoided.
- Specimen must be accompanied by a laboratory requisition giving the patient's name, age, unique identifier, clinician, clinical history, procedure location, specimen type submitted, time and date.
- The reason for referral is required with specimen submission.

Collection: Prepared Slide – ThinPrep®

- Process slide according to your ThinPrep procedure.
- Fix in Carnoy's Solution (1 part acetic acid to 3 parts methanol) for 15 minutes.
- Air dry and submit with a laboratory requisition giving the patient's name, age, unique identifier, clinician, clinical history, procedure location, specimen type submitted, time and date.
- The reason for referral is required with specimen submission.